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 **AccuVein<sup>®</sup>**

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Guidance and Manufacturer's Declaration			
The AV400, CC300, and HF470 are intended for use in the electromagnetic environment specified below. The customer or the user of the AV400, CC300, or HF470 should assure that it is used in such an environment.			
IMMUNITY Test	IEC 60601 Test Level	Compliance Level	Electromagnetic environment - guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst EC 61000-4-4	± 2 kV for power supply lines ± 1 kV input/output lines	± 2 kV for power supply lines Not applicable	Mains power quality should be that of a typical commercial or hospital environment
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV differential mode ± 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% $U_n$ (>95% dip in $U_n$ ) for 0.5 cycles 40% $U_n$ (60% dip in $U_n$ ) for 5 cycles 70% $U_n$ (30% dip in $U_n$ ) for 25 cycles <5% $U_n$ (>95% dip in $U_n$ ) for 5 s	<5% $U_n$ (>95% dip in $U_n$ ) for 0.5 cycles 40% $U_n$ (60% dip in $U_n$ ) for 5 cycles 70% $U_n$ (30% dip in $U_n$ ) for 25 cycles <5% $U_n$ (>95% dip in $U_n$ ) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the AV400 requires continued operation during power mains interruptions, it is recommended that the AV400 be powered from an uninterruptible power supply or a battery
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE: $U_n$ is the AC mains voltage prior to application of the test level.			

Recommended separation distances between portable and mobile RF communications equipment and the AV400			
The AV400, CC300, and HF470 are intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the AV400, CC300, or HF470 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the AV400 as recommended below, according to the maximum output power of the communications equipment.			
Rated Maximum Output power of transmitter $W$	Separation distance according to frequency of transmitter $m$		
	150 kHz to 80 MHz $d = 1.2\sqrt{P}$	80 MHz to 800 MHz $d = 1.2\sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance $d$ in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where $P$ is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.			

Guidance and Manufacturer's Declaration - Electromagnetic Emissions		
The AV400, CC300, and HF470 are intended for use in the electromagnetic environment specified below. The customer or the user of the AV400, CC300, or HF470 should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic Environment Guidance
RF Emissions CISPR 11	Group 1	The AV400 uses RF energy only for its internal function. Therefore, its RF emissions are very low and not likely to cause any interference in nearby electronics
RF Emissions CISPR 11	Class A	The AV400 is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes
Harmonic Emissions IEC 6100-3-2	Complies	
Voltage Fluctuations IEC 6100-3-3	Complies	

Guidance and Manufacturer's Declaration - Electromagnetic Immunity			
The AV400, CC300, and HF470 are intended for use in the electromagnetic environment specified below. The customer or the user of the AV400, CC300, or HF470 should assure that it is used in such an environment.			
IMMUNITY TEST	IEC 60601 TEST LEVEL	COMPLIANCE LEVEL	Electromagnetic Environment - Guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the AV400, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1.2\sqrt{P}$
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 Vrms	$d = 1.2\sqrt{P}$ 80 MHz to 800 MHz $d = 2.3\sqrt{P}$ 800 MHz to 2.5 GHz  where $P$ is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and $d$ is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol:
			

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.  
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the AV400 is used exceeds the applicable RF compliance level above the AV400 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the AV400. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

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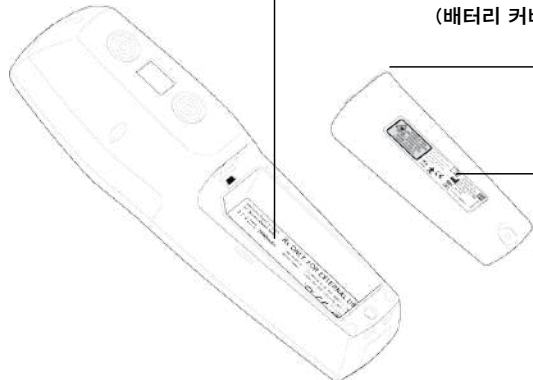
Manufactured in USA for AccuVein Inc. **Rx ONLY. FOR EXTERNAL USE.**  
 Use BA400 battery. Conforms to UL Std. 60601-1; Cert. to CAN/CSA Std. C22.2 No. 601.1-M90  
**3.7 V 3100mAh**  
**REF AV400 Rev. 1.0**  
**SN AV12401234**  
 ETL CLASSIFIED  
 Limited Warranty  
 Patient E: 0608750  
 CE  
 DC85  
 0004744A

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**EC** EMERGO EUROPE  
 Prinsesgracht 20  
 2514 AP, The Hague  
 The Netherlands  
 Phone: +31.70.345.8570  
 Fax: +31.70.346.7299

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**VISIBLE AND INVISIBLE LASER RADIATION.**  
**DO NOT STARE INTO BEAM. CLASS 2 LASER PRODUCT.**

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
Conforms with 21 CFR 1040.10 and 1042.11 except for Jewellers  
 dated June 24, 2007

**Rx Only**

AccuVein Inc.  
 40 Zoube Mill Rd  
 Cold Spring Harbor  
 New York 11724 USA  
 816.957.9400

**REF AV400** **SN AV12401234**

0004744A

AccuVein Inc.

40 Goose Hill Road

Cold Spring Harbor, NY 11724

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[www.accuvein.com](http://www.accuvein.com)

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© AccuVein Inc. 2012